

Latex or Nitrile Gloves

HOW TO WEAR

The key when dealing with gloves and putting them on is cleanliness. Always keep cleanliness in mind to ensure gloves are being used optimally. Apply gloves in a clean area, and always with clean hands. Do not contaminate the gloves by placing them on a dirty surface.

APPLICATION

To preserve cleanliness, attempt to not touch the outside of the glove while putting them on. First put the glove on the dominant hand, touching it as little as possible. Then put on the other hand, using the gloved hand to pull it on. Once both gloves are on, adjust the fit.

REMOVAL

Beginning with the dominant hand, remove the other glove by peeling it off without touching the outside. Then proceed to do the same thing with the other hand, carefully using the un-gloved hand to remove the other glove.

KEEP IN MIND

The most important thing to remember when removing gloves is to not touch the outside, to avoid the transfer of unwanted liquids or substances to the skin.



Cloth Face Masks

Wash after use

APPLICATION

Before putting cloth face coverings on, clean hands with alcohol-based hand rub or soap and water. Cloth face coverings should be secured with ties or ear loops, and fit snugly but comfortably.

REMOVAL

When removing a cloth face covering, individuals should be careful not to touch their eyes, nose, or mouth during removal. Wash hands immediately after removing.

For those contemplating the CDC's recommendation to wear cloth face coverings, CDC would like the following taken into consideration:

- Cloth face coverings should not be placed on young children under age 2, anyone who has trouble breathing, or is unconscious, incapacitated or otherwise unable to remove the mask without assistance.
- Cloth face coverings should allow for breathing without restriction.
- Cloth face coverings should be routinely washed and dried without damage or change to shape.
- A washing machine should suffice in properly washing a face covering.




www.mtmtransit.com



Personal Protective Equipment


Proper Wear and Disposal

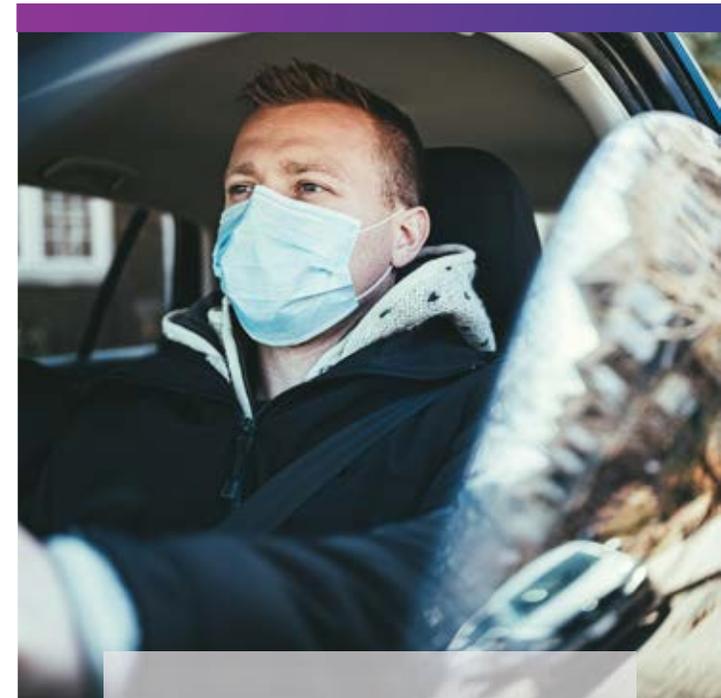

KN95 3D Protective Respirator

Disposable Ear-Loop Face Masks

Latex or Nitrile Gloves

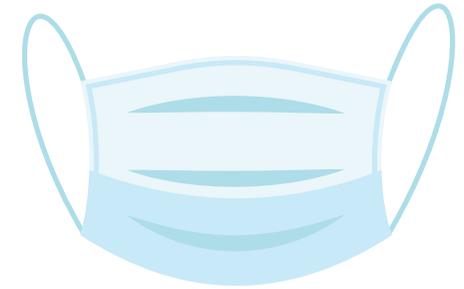

KN95 3D Protective Respirator

Expected lifespan is 5 days unless soiled

HOW TO WEAR

Before putting it on, clean hands with alcohol-based hand rub or soap and water. Always apply the mask with clean hands.

APPLICATION

Hold the respirator in hand, the fingertip located at the nose clip. Put the respirator over the face, and pinch the metal strip or stiff edge of the mask so it takes the shape of the nose bridge. Pull the ear-bands behind the ear, and adjust the whereabouts of the nose strip to ensure tightness.

REMOVAL

Remove mask from behind (do not touch the front of the mask!). Discard immediately into a closed bin, if available. Clean hands with alcohol-based hand rub, or soap and water.

WEARING TEST

Cover the mask with hands and exhale. If there is air leaking from nose location, tighten the nose clip. If air leaks from the edge, re-adjust the ear-band to ensure tightness.

Note

To ensure that the respirator is clean and hygienic, avoid touching the respirator. Wash hands before and after touching the respirator each time.

This respirator is maintenance-free (Do not wash it or use a microwave oven to heat it). When the filter cotton is seriously dirty, damaged, or breathing resistance increases significantly, replace the respirator.

Disposable Ear-Loop Face Masks

Throwaway after use

HOW TO WEAR

Before putting it on, clean hands with alcohol-based hand rub or soap and water. Always apply the mask with clean hands.

APPLICATION

Ensure the proper side of the mask faces outwards, with the metal strip at the top. Hold the mask via ear loops, ensure that it covers the mouth and nose, and place the loops over the ears. Pinch the metal strip or stiff edge of the mask so it takes the shape of the nose bridge.

REMOVAL

Remove mask from behind (do not touch the front of the mask!). Discard immediately into a closed bin, if available. Clean hands with alcohol-based hand rub, or soap and water.

Note

These are not surgical masks